IMPORTANT / BELANGKRIJK

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You will find your test result on www.myhealth.belgium.be approximately 2 weeks after sending it. Your general practitioner will receive a copy.

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If the test is **NEGATIVE**, you will receive **—** another Colotest in 2 years time.

If the test is **POSITIVE**, contact your general practitioner

They will prescribe a colonoscopy to check where the bleeding is coming from.

THIS CHECK-UP IS NECESSARY but don't be worried, 9 times out of 10, it's not cancer

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Advena Ltd.
Tower Business Centre, 2nd Flr., Tower Street,
Swatara, BKR 4013 Malta

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Colorectal cancer screening programme
Screeningstest voor Darmkanker

A QUICK AND SIMPLE TEST THAT CAN SAVE YOUR LIFE

In Belgium, **8.700 cases of colorectal cancer** (bowel cancer) are diagnosed each year.

If detected at an early stage, colorectal cancer is **curable in 90% of cases**. For this reason, it is recommended that a faecal occult blood test is carried out every two years, **from the age of 50 until 74**. This screening test detects blood in the stool.

This test is not suitable for you if

- you have a chronic inflammatory bowel disease
- you or at least one first-degree relative (father, mother, brother, sister) has had colon cancer
- you have bleeding haemorrhoids

If you find yourself in one of these situations, talk to your doctor first.

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COMPLETE THE IDENTIFICATION DOCUMENT WITH (IN UPPER CASE)

- An health insurance fund **identification sticker** ;
- your **National Register number** (on the back of your ID card);
- the **date you performed** your test;
- your full **personal information**;
- your **general practitioner's contact details**, in full and legible.

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Check the expiration date on your test. **After this date, the test is no longer usable.**

Write your name (N), age (A) and the date you performed the test (D) on the test in the designated area. Please do not stick anything on the test and do not remove the barcode.

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Unfold the stool collection bag using the arrows.

Stick it to your toilet seat, according to the picture or opt for a plastic bag.

Important: for the test to be successful, the stool must not come into contact with any liquid (urine, bleach, etc.)

Once the collection bag is installed, have a bowel movement.

Take the test and open it by turning the green cap.

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
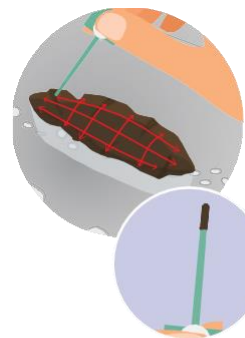
Pass the sampling stick over the surface of your stool as shown in the picture. Completely cover the grooves of the sampling stick, without excess.

Note: the stool collection paper is biodegradable and compatible with septic tanks.

Place the sampling stick back in the tube and press the green cap firmly until you hear a "click". Shake the tube to allow the liquid inside to mix with the stool.

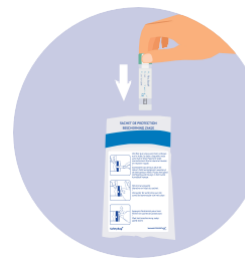After closing, do not reopen the tube.

Place your test in the **green protective bag**. Close it.

Keep the test in the refrigerator until you send it.

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3 Sending the test / Verzenden van de test

Check all the information on the identification document.

Place the identification document and the green protective bag containing the test into the pre-addressed return envelope with the UN 3373 logo (grey envelope). Close the envelope and post it without a stamp.

Do not use the blue envelope !

Mail the envelope on a working day **no later than 5 days** after the sample collection (not the day before a weekend or a public holiday).

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